ISTRUZIONI D'USO

Italiano

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INSTRUCTIONS FOR USE

English

Tray Adhesive for A-Silicones ensures perfect adhesion between addition silicone-based impression materials (PVS) and minimally or non-retentive impression trays (metal, plastic or resin). Please check compatibility with your plastic tray.

APPLICATION

1. Use the brush to apply a fine, regular layer of adhesive onto the internal surface of the impression tray before applying the impression material.
The impression tray must be perfectly clean, grease-free and dry.
2. Allow to dry for 3 minutes, you then have 30 minutes at maximum to apply the silicone.

PRECAUTIONS FOR USE

- Do not swallow
- Highly flammable
- Do not breathe the vapour
- Avoid contact with skin and eyes

SIDE EFFECTS

- There are no known systemic side effects to date, however,
- do not use in cases where known allergies to the components exist
 - in some cases individual contact allergies may exist

COMPOSITION

Silicone resin, solvent (volatile paraffines).

STORAGE

Store at room temperature ($\leq 77\text{ }^{\circ}\text{F}/25\text{ }^{\circ}\text{C}$).

Do not use after expiration date. Close the bottle tightly after use.

DISPOSAL

Please dispose in accordance with local regulations.

PACKAGING

1 bottle @ 10 ml liquid

REF 999394

Caution: Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

Keep away from children! For dental use only!

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